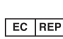
	
<h1>EPICYN™</h1>	
	
Oculus Technologies of México S.A. de C.V. Industria Vidriera No. 81, Industrial Zapopan Norte Zip Code 45130, Zapopan, Jalisco, México. Phone: + 52 33 38 33 67 22	
	
Sonoma Pharmaceuticals Netherlands B.V. (Formerly Oculus Innovative Sciences Netherlands B.V.) Boven de Wolfskuil 3, C30-C32 6049 LX Hertfen/Roermond The Netherlands Phone: +31 (0)475 318 666 Fax: +31 (0)475 318 342 www.sonomapharma.eu	ICCRD-009 Rev. 05 June 10 th 2021 EOCAC-691 ED-09

EPICYN™

Instructions for use

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Istruzioni d'uso

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Bruksanvisning

Bruksanvisning

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Kasutusjuhend

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EPICYN™

Fragile, handle with care

Fragile

Frágil, manipular con cuidado

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Ömtåligt, handskas varsamt

Frágil, manusear com cuidado

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Instructions for use:

Indications for use: Epicyn™ is indicated as an adjuvant in the wound healing process with wounds that can only heal by secondary intention in maturation phase. Epicyn™ is effective for the management and reduction of new and existing hypertrophic and keloid scars. Epicyn™ assists with moistening while forming a protective barrier against physical, chemical and microbial invasion of the scar.

Handling: Epicyn™ is biocompatible for its indicated use, non-irritating to human skin and is not a skin sensitizer. No special handling precautions are required.

Storage: Epicyn™ should be stored in its original sealed container at ambient conditions. Store at room temperature and prevent product from freezing. It is non-flammable. After use, the container should be closed again, before storing.

Stability: Stability of the unopened product is assured from date of manufacture until expiration (see product’s label for expiration date). Upon opening the stability is assured up to 60 days after opening. Upon opening note the date on the bottle in the section “Date Opened”.

Directions for use: Epicyn™ is ready for use with no mixing or dilution required. Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Apply three times daily.

Treatment: Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Wash the affected area as recommended by your healthcare professional. Dry. Cover the affected area with a thin layer of Epicyn™ and gently rub until it is evenly distributed. Epicyn™ can ideally be combined with any preferred dressing.

Warnings and points to note: <ul style="list-style-type: none">•If tamper resistant seal is broken, do not use. •For external use only, not for injection. Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. •Do not use if sensitive to polydimethylsiloxane. •Do not use Epicyn™ in combination with other topical products without the advice of a healthcare professional. •In case of deep or serious wounds or if you have any other treatment concerns consult a healthcare professional. •If signs of infection consult a healthcare professional. •Report any adverse reactions to a healthcare professional. •Keep out of the reach and out of a sight of children.

Disposal: No special disposal considerations required. Follow local regulation for unused and expired pharmaceutical products.

Ingredients: Electrolysed Water, Polydimethylsiloxane, Sodium Magnesium Fluorosilicate, Sodium Phosphate, ancillary substances: Sodium Hypochlorite and Hypochlorous Acid.

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